



QUET ECG -APP
TEST VERSION
30/9/2024

FIRST STEPS AND QUICK GUIDE FOR VETERINARIANS



www.quetecg.com

+39 3757192413

hello@quetecg.com

01 DOWNLOAD AND
REGISTRATION

02 LOGIN AND MENU

03 RECORD

04 MONITOR

05 PATIENT EVENTS
AMPLITUDE & SPEED

06 HISTORY

07 ECG DOWNLOAD
& REPORTS

08 DASHBOARD

01 DOWNLOAD AND REGISTRATION




MAKE SURE YOU HAVE A QUET ECG DEVICE (Q1 OR Q6). APP IS COMPATIBLE WITH EKUORE ECG DEVICES.

DOWNLOAD QUET ECG APP ON YOUR ANDROID SMARTPHONE OR TABLET FROM PLAY STORE (VERSION FOR IOS SOON AVAILABLE)



REGISTER AS A VETERINARIAN

IF YOU HAVE ALL REQUIREMENTS, WE'LL MAKE SURE YOUR ACCOUNT IS APPROVED IN A FEW HOURS.

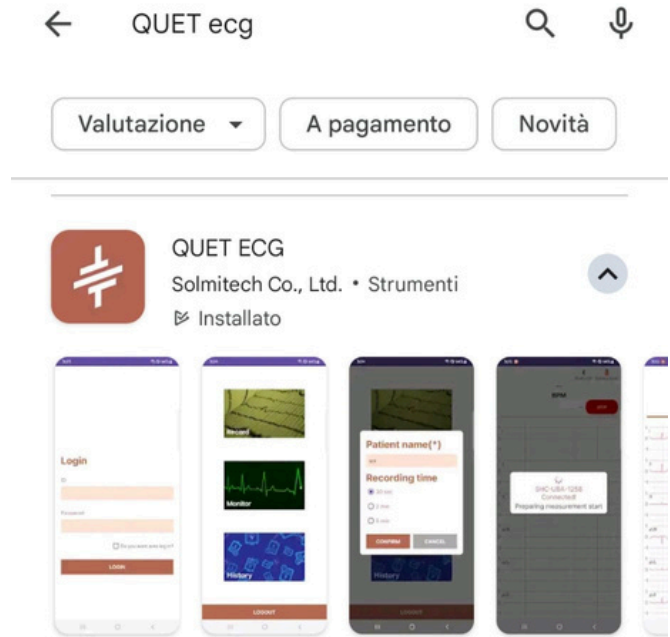

Accedi

ID

Password

Vuoi rimanere connesso?

ACCEDI



Register



Contact us

02 LOGIN AND MENU



OPEN APP AND LOGIN USING YOUR ID AND PW.

MAKE SURE TO FLAG THE "STAY CONNECTED" OPTION



Accedi

ID
01

Password
....

Vuoi rimanere connesso?

ACCEDI



APP MENU IS MADE OF 3 PARTS:

- **RECORD** (30 SEC - 2MIN- 5 MIN 10 MIN ECGS WITH PDF PRODUCTION)
- **MONITOR** (ECG ON SCREEN AND RECORDED)
- **HISTORY** (PAST ECGS FOLDER + ECG REPORTS PANEL AND DOWNLOAD)



03 RECORD



TO RECORD YOUR PATIENTS' ECGS :

- CLICK RECORD
- INPUT PATIENT NAME AND SELECT ECG DURATION
- TURN ON YOUR ECG DEVICE



ONCE CONNECTION IS ESTABLISHED, ECG TRACING WILL APPEAR ON SCREEN.

AT THE END OF RECORDING, YOU WILL BE ASKED WHETHER YOU WANT TO SEND IT TO QUET ECG REPORTING CENTER



NO> BACK TO MAIN MENU (PDF AVAILABLE IN "HISTORY")

YES>YOU WILL BE ASKED TO INPUT YOUR PATIENT SHORT ANAMNESIS AND YOUR ECG REPORT WILL SOON BE AVAILABLE IN YOUR "HISTORY"



Nome paziente(*)

Durata registrazione

30 sec


2 min

5 min

CONFERMA ANNULLA



miki --
2024.09.23 10:04:40
(00:00:29) **BPM**

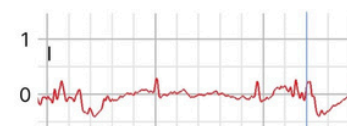

Vuoi inviare la registrazione ad un centro di refertazione?

Si No

04 MONITOR



FOR LONGER MONITORING OF YOUR PATIENTS' ECG:

- CLICK MONITOR IN MENU MENU
- INPUT PATIENT NAME
- TURN ON ECG DEVICE
- SEE ECG ON SCREEN





TO STOP MONITORING, CLICK "FINISH" RED BUTTON ON TOP RIGHT CORNER.

YOU WILL BE ASKED IF YOU WANT TO SEND ECG TRACING TO QUET CARDIOLOGIST AND TO INSER PATIENT DATA



ECG IS REVIEWABLE IN YOUR "HISTORY" SECTION

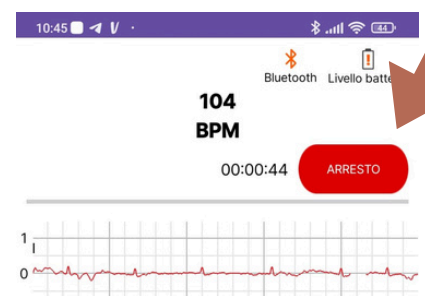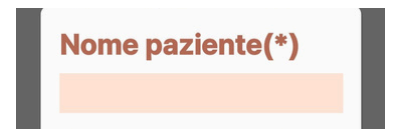


05

PATIENT EVENTS, AMPLITUDE & SPEED CHANGE



TO ADD A SYMPTOM OR PATIENT EVENT WHILE RECORDING CLICK “SYMPTOM REPORT” ON TOP LEFT CORNER.

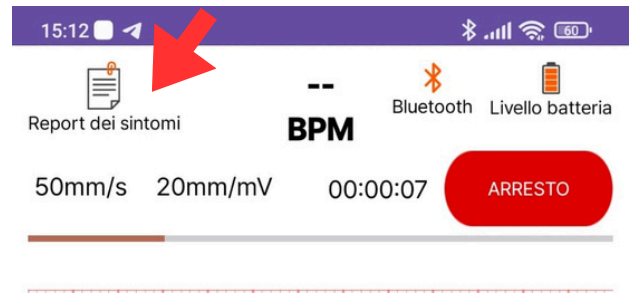

YOU WILL BE ASKED TO SELECT A SYMPTOM AND AN EVENT. THE REPORTING CARDIOLOGIST WILL SEE THOSE ASSOCIATED WITH THE ECG TRACING



TO CHANGE VOLTAGE AND PAPER SPEED, JUST CLICK ON THE VALUES



50mm/s 20mm/mV




50mm/s 10mm/mV

06 HISTORY

ALL ECGS RECORDED WITH YOUR DEVICE (SMARTPHONE OR TABLET) ARE STORED IN THIS SECTION.

ECG ARE IN CHRONOLOGICAL ORDER, WITH PATIENT NAME AND RECORDING TIME.

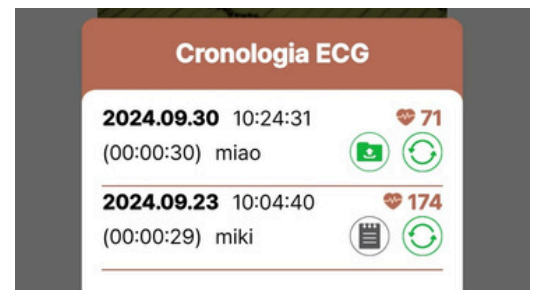

ICONS LEGENDA

Data has not uploaded (Click icon to upload)

Data uploaded Under reviewing

Finish reviewing Report available (Click icon to see report)

ECG report (Show report in app)


07 ECG DOWNLOAD AND REPORTS



TO DOWNLOAD ECG IN PDF, GO TO "HISTORY" AND SELECT



WE SUGGEST TO CREATE A SPECIFIC FOLDER ON YOUR SMARTPHONE/TABLET TO ALWAYS FIND ECGS IN ONE PLACE

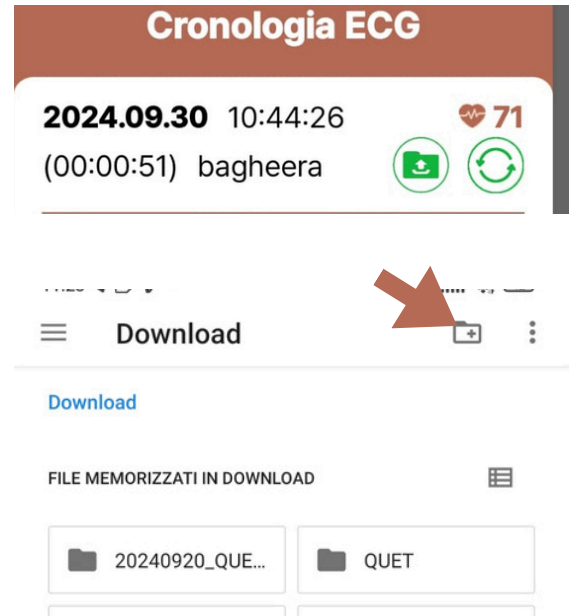

TO DOWNLOAD YOUR ECG REPORTS PDF, GO TO "HISTORY" AND CLICK GREEN ICON



08 DASHBOARD











YOU CAN HAVE ACCESS TO ALL YOUR ECG RECORDINGS AND ECG REPORTS FROM HERE.

JUST LOGIN WITH YOUR VET ID AND PASSWORD

 
My Page Log Out

Veterinarian ID : vet

Client	Patient ID	Patient	Species	Lineage	Age(Yr)	Gender	Weight(Kg)	Note	Device ID	Start	Finish	Duration	ECG Data	Report	Report Upload	Delete
bassi	008	miki	Cane		0	nessuno	0	View	SHC-U7D-2228	2024-09-23 10:04:40	2024-09-23 10:05:09	0:00:29	Download	View	2024-09-30 18:44:03	
miki	a02	furia	Cavallo	baio	0	nessuno	0	View	SHC-U7D-2228	2024-09-20 09:56:11	2024-09-20 09:59:14	0:03:03	Download	View	2024-09-20 17:30:02	
francesca	b01	baggy	Cane	labrador	0	nessuno	0	View	SHC-U7D-2228	2024-09-20 09:53:45	2024-09-20 09:54:15	0:00:30	Download	-	-	
IO	123	BOB	Gatto		6	Maschio	7	View	SHC-UB-1474	2024-09-18 15:16:22	2024-09-18 15:16:52	0:00:30	Download	View	2024-09-20 17:31:52	
Mimmo	Stalla2	Tempest	Cavallo		2	Femmina	200	View	SHC-UB-1474	2024-09-18 14:42:08	2024-09-18 14:43:10	0:01:02	Download	-	-	
Pecco	123	Bob	Cane	Carlino	3	Maschio	15	View	SHC-UB-1474	2024-09-18 14:36:24	2024-09-18 14:36:54	0:00:30	Download	View	2024-10-01 18:14:40	
Quet	123	luca	Cane	labrador	8	Maschio	45	View	SHC-UB-1474	2024-09-18 11:53:24	2024-09-18 11:53:54	0:00:30	Download	-	-	
michela	30	michela	Altro		39	Femmina	0	View	SHC-U7D-2228	2024-09-09 12:21:06	2024-09-09 12:21:38	0:00:32	Download	-	-	
Luca	G01	Rufi	G	Bri	7	Maschio	4	View	SHC-UB-1459	2024-07-15 13:28:52	2024-07-15 13:29:04	0:00:12	Download	-	-	
Luca	G01	Rufi	Gatto	british	7	Maschio	4	View	SHC-UB-1459	2024-07-15 12:42:47	2024-07-15 12:43:55	0:01:08	Download	-	-	
p	4	I	equi	c	12	Maschio	200	View	SHC-UB-1459	2024-07-05 13:07:14	2024-07-05 13:07:44	0:00:30	Download	View	2024-07-05 20:29:39	

TO DISCOVER MORE ON ECG DEVICES & REPORTS CONTACT US AT

www.quetecg.com

[+39 3757192413](tel:+393757192413)

hello@quetecg.com

DON'T FORGET TO CHECK TO ALWAYS HAVE THE UPDATED VERSION OF OUR APP !